

USER MANUAL for Large Pet Scale 7474 / 7475



Before using the Scale

- The Scale consists of a plastic platform. There are three easy to use function keys: **ON/OFF/TARE, HOLD, UNIT.**
- The scale can be used in a fixed location or as a portable scale, powered by 2 x AAA batteries.
- Before use, ensure the scale is on a flat surface. An un-even surface for weighing could result in inaccurate readings.

Operation

Switching the Scale on/off

- To switch the Scale on, press **ON/OFF/TARE.**
- When the scale shows a line of zeros, it is ready to use.
- To switch the Scale off, press and hold **ON/OFF/TARE.**

Using the Hold function

- The Hold function allows you to obtain an accurate weight reading, even if the animal is moving on the scale.
- With the animal on the scale, press HOLD once.
- The weight reading will flash several times, and then display the correct weight for around six seconds.
- Please note: The scale will automatically power-off after around 6 seconds to save battery power.

Using the Tare function

- The Tare function allows you to remove unwanted weight from the display, such as a towel or blanket.
- With the scale switched on, place the item you wish to tare off onto the platform.
- Press ON/OFF/TARE once. The display will show zero.
- You can now place your pet on the scale with the towel or blanket, and only the pets weight will display.
- Switching the scale off will cancel the tare weight.



Using the Unit function

- To change the unit of measure, press the **UNIT** button with the animal on the scale.
- The weight on the display will now change between kg and lb.

Cleaning

- Please do not use corrosive liquids or large amounts of water to clean the scale. We recommend using alcohol-based wipes or similar.
 - Always disconnect the scale from the main power supply before cleaning.
- ⚠ Never disassemble the scale as this may cause malfunction.

Error Messages

Display: **Err**

Problem: **Overload**

Solution: If the total load exceeds 150kg, the display will show this message.

Product Specification

Capacity/Division: 150kg x 100g

Units of Measure: Kg/lb

Function Keys: ON/OFF/TARE, HOLD, UNIT

Display: Easy to read LCD display with four active digits

Operating: Temperature 5 to 30°C

Power Supply: 2 x AAA batteries